MOPEDIA®

MA RC200-45_RC210-45 02 A_ITA_07 2020

SEDIE COMODE

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ITALIANO

INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

NOTE

ITALIANO

MOPEDIA®

MA RC200-45_RC210-45 02 A_ENG_07 2020

COMMODE CHAIRS

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES	PAG.3
2. INTRODUCTION	PAG.3
3. INTENDED USE	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. WARNINGS	PAG.4
5.1 General warnings	pag. 4
5.2 Warnings for use.....	pag. 4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION	PAG.5
8. ASSEMBLING	PAG.5
8.1 Legrest adjustment (where existing).....	pag. 5
9. BEFORE USE	PAG.5
10. WARNING FOR USE	PAG.5
10.1 Brakes.....	pag. 6
10.2 Wheels	pag. 6
11. HOW TO USE	PAG.6
11.1 Get in the commode chairs	pag. 6
11.2 Get out of the commode chairs	pag. 6
12. MAINTENANCE	PAG.6
13. CLEANING AND DISINFECTION	PAG.7
13.1 Cover and padding.....	pag. 7
13.2 Toilet cleaning	pag. 7
13.3 Frame and metallic parts	pag. 7
13.4 Brakes and wheels	pag. 7
13.5 Disinfection.....	pag. 7
14. CONDITIONS OF DISPOSAL	PAG.7
15. TECHNICAL FEATURES	PAG.7
15.1 Dimensions and weight	pag. 7
15.2 Technical specification.....	pag. 7
16. SPARE PARTS/ACCESSORIES	PAG.8
17. WARRANTY	PAG.8
18. REPAIR	PAG.8
18.1 Warranty repair	pag. 8
18.2 Repair not covered by warranty.....	pag. 8
18.3 Non-defective device.....	pag. 8
19. SPARE SPARTS	PAG.8
20. EXEMPT CLAUSES	PAG.8

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RC200-45

Commode chair with 4 adjustable legs - 45 cm

RC210-45

Commode chair on castors with swing away legrest - 45 cm

2. INTRODUCTION

Thank you for purchasing a MOPEDIA commode chair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The commode chairs are an aid for evacuation and mobility for people with walking difficulties.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of COMMODE CHAIRS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. WARNINGS

5.1 General warnings

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the bath chair
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

5.2 Warnings for use

- Make sure the tank has not been moved or removed
- Do not lean on the legrests to sit down or get off the commode chair
- Do not lean out from the commode chair to reach items on furniture or on other shelf
- Avoid sitting on the edge to prevent the convenient one can overturn

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

7. GENERAL DESCRIPTION

RC200-45 Seat L 45x42 W cm
4 LEGS HEIGHT ADJUSTABLE

RC210-45 Seat L 45x42 W cm
4 CASTORS SWING AWAY LEGREST

8. ASSEMBLING

All the MOPEDIA commode chair are provided with backrest and footrest (where present) disassembled. For the assembly of the backrest is sufficient to put the two ends of the tubes into the openings provided in the back of the chair. Then fix it backrest using clips provided. The platforms must be placed outside the comfortable chair by inserting two pins in the upper part of the platform. The footrest must always be within the comfortable chair. To secure the platform to make sure that the lever. Locking is secure within the comfortable chair. are provided with backrest and footrest (where present) disassembled. For the assembly of the backrest is sufficient to put the two ends of the tubes into the openings provided in the back of the chair. Then fix it backrest using clips provided. The platforms must be placed outside the comfortable chair by inserting two pins in the upper part of the platform. The footrest must always be within the comfortable chair. To secure the platform to make sure that the lever. Locking is secure within the comfortable chair.

8.1 Legrest adjustment (where existing)

The legrest of the commode chairs can be adjusted according to the height that offers the best comfort loosening the screw. Always check that the two platforms are at same height.

9. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

10. WARNING FOR USE

- The commode chair is for internal use only, can not be used as a seat on motor vehicles;
- To push the commode chair, always use only the rear handle, Never push using other supports;
- Before moving on an inclined surface to make sure of their abilities and limitations. Consult movement techniques with qualified personnel;
- It 'still recommended care assistance for land with a slope greater than to 10%. If you need to stop on an inclined surface, avoid sudden movements;
- If two people raise the commode chair, avoid contesting in points dangerous for the patient as footrests and armrests, but use only the frame and rear handles.

10.1 Brakes

Lock the wheels of the chair before you ascend, descend, stretch forward. On the model to small wheels (RC210-45) the brakes are operated by pressing the lever on all wheels of the chair (Fig. 1).

Fig. 1

10.2 Wheels

The front wheels must be periodically checked and lubricated if necessary.

11. HOW TO USE

Note: make reference to "GENERAL WARNINGS" chapter

WARNING!

- Getting in and out of the commode chair may be performed independently, provided you have a adequate mobility and upper body strength
- Do not stand on removable parts DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE COMMODE CHAIR, OVERTURNING RISK!!!

11.1 Get in the commode chairs

1. Position the commode chair as close as possible to you
2. Engage wheel locks (only for models with castors). We suggest to remove the armrest to ease the patient getting in the commode chair
3. Lean on the frame with arms and slowly go down towards the seat

11.2 Get out of the commode chairs

1. Position the commode chair as close as possible along side the seat to which you are transferring
2. Engage wheel locks (only for models with castors). We suggest to remove the armrest to ease the patient getting out of the commode chair
3. Shift body weight into seat with transfer

12. MAINTENANCE

The MOPEDIA devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter. Moretti MOPEDIA commode chairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically.

Element	Frequency
Braking capacity	Monthly
Seat cleaning	As needed
General inspection by qualified personnel	Biennial

13. CLEANING AND DISINFECTION

13.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

13.2 Toilet cleaning

The toilet must be regularly cleaned and disinfected using

13.3 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

13.4 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

13.5 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

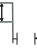
Note: Do not use solvent and abrasive product

14. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

15. TECHNICAL FEATURES

15.1 Dimensions and weight

Dimensions and weight													
Measures in CM													
RC200-45	-	55	42	95÷105	74÷84	53÷63	-	45	42	46	11,5	-	-
RC210-45	89	56		95	74	53	46÷54			49	12,5	12,5	12,5

15.2 Technical specification

FRAME	POWDER COATED STEEL
FABRIC	PVC
Seat	PLASTIC
 MAX WEIGHT	120 KG
ENVIRONMENTAL TEMPERATURE	- 20 C° / + 40 C°
STORAGE	STORE IN A COOL PLACE, AWAY FROM HEAT SOURCES, SUCH AS SUNLIGHT (SUGGESTED TEMPERATURE BETWEEN +10°C AND +40°C)
WHEELS	Ø 12,5 CM

16. SPARE PARTS/ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

17. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product , and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

18. REPAIR

18.1 Warranty repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty.

Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

18.2 Repair not covered by warranty

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

18.3 Non-defective device

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

19. SPARE SPARTS

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

20. EXEMPT CLAUSES

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

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