MOPEDIA®

MA RI900_RI905 02 A* ITA_02-2023

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Via _____ Località _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com

MOPEDIA®

MA R1900_R1905 02 A* ENG_02-2023

**MUEVO HOME - MUEVO HOME LIGHT
LIFT AND TRANSFER**

INSTRUCTION MANUAL

ENGLISH

INDICE

1. CODES	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.5
7.1 Description of the parts of RI900	pag. 5
7.2 Description of the parts of RI905	pag. 6
7.3 Description of the parts of RIA910 accessory arm.....	pag. 7
8. ASSEMBLY INSTRUCTIONS	PAG.7
8.1 RI900-RI905 assembly instructions.....	pag. 7
8.2 RIA910 accessory Assembly instructions	pag. 8
8.3 Assembly instructions for slings.....	pag. 8
9. INSTRUCTIONS FOR USE	PAG.9
9.1 Instructions for use of the RI900-RI905 patient lift	pag. 9
9.1.1 Specific instructions for RI900 Opening/Closing legs	pag. 10
9.1.2 Specific instructions for RI905 Opening/Closing legs	pag. 10
9.2 Instructions for use of RI900-RI905 patient lift with RIA910 lifting arm.....	pag. 10
9.3 Feature Details	pag. 11
9.4 Instructions for using the remote control	pag. 12
9.4.1 RI900 Remote control.....	pag. 12
9.4.2 RI905 Remote control.....	pag. 13
10. WARNINGS.....	PAG.13
11. CLEANING	PAG.13
12. MAINTENANCE.....	PAG.14
13. SPARE PARTS AND ACCESSORIES.....	PAG.14
14. TECHNICAL FEATURES.....	PAG.14
15. CONDITIONS OF DISPOSAL.....	PAG.16
15.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE	pag. 16
15.2 Treatment of dead batteries (Directive 2006/66/CE)	pag. 16
16. WARRANTY	PAG.16
16.1 Warranty for rechargeable batteries (if provided)	pag. 16
16.2 Reparations	pag. 17
16.3 Spare parts.....	pag. 17
16.4 Exoneration clauses	pag. 17

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

RI900 - Electrical lift Muevo Home - payload 180 Kg

RI905 - Electrical lift Muevo Home Light - payload 135 Kg

2. INTRODUCTION

Thank you for choosing a patient lift and transfer device from the **MUEVO HOME by MOPEDIA** product range by MORETTI S.p.A.. Their design and quality are a guarantee of comfort, safety and reliability. **MUEVO HOME by MOPEDIA** patient lifts/stand up lifts by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advice for your safety. Please read carefully through the manual before using the patient lift/stand up lift. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

MUEVO HOME by MOPEDIA patient lifts are intended to lift and move people with motor difficulties. An assistant operates the machine. The lifts cannot be used by only one person. MORETTI S.p.A. refuses all liability for damages caused by improper use of the device or by its use for a purpose other than that one specified in this manual.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PATIENT LIFTS MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017.

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels

5. GENERAL WARNINGS

WARNING!

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual, checking with a particular attention at the warnings before use.
- For proper use of the device to consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Conditions of disposal

Attention, should read the instructions

Production dates

Waste Electrical & Electronic Equipment (WEEE)


Protection against dust and water splashes

B Type

Max loading weight

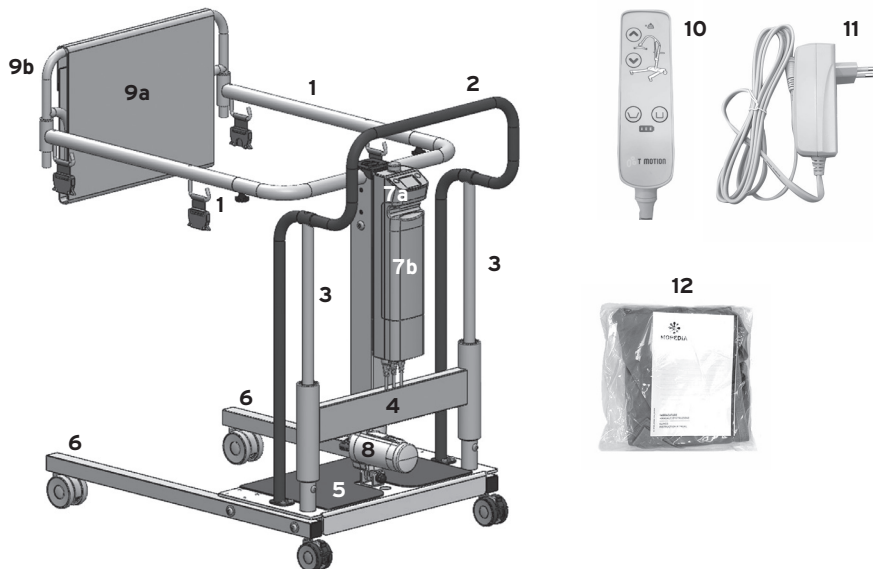7. GENERAL DESCRIPTION
7.1 Description of the parts of RI900

Images for illustrative purpose only

1. n.2 Removable arms equipped with attachment points for harness with locking handwheel
2. n.1 handle for movement
3. n.2 Sliding columns for lifting
4. n.1 Central body (sliding groups with bearings, support beam, support column, half-arms with handle for patient)
5. n.1 Basement equipped with:
 - n.1 Non-slip Mat (basement upper part)
 - n.1 castor \varnothing 65mm (basement lower part)
 - n.2 Actuators for leg opening-closing movement (basement lower part)
 - n.4 sliding groups for legs opening-closing movement (basement lower part)
6. n.2 Legs equipped with:
 - n.2 castors \varnothing 75mm without brake (front part)
 - n.2 castors \varnothing 65mm with brake (rear part)
7. n.1 Control unit (7a) with removable battery (7b)
8. n.1 Lifting actuator
9. n.1 backrest (9a) with n.1 backrest padding (9b)
10. n.1 Remote control
11. n.1 Battery charger
12. n.1 RIA201 Standard harness/seat complete with straps with buckle and loop for height adjustment

7.2 Description of the parts of RI905


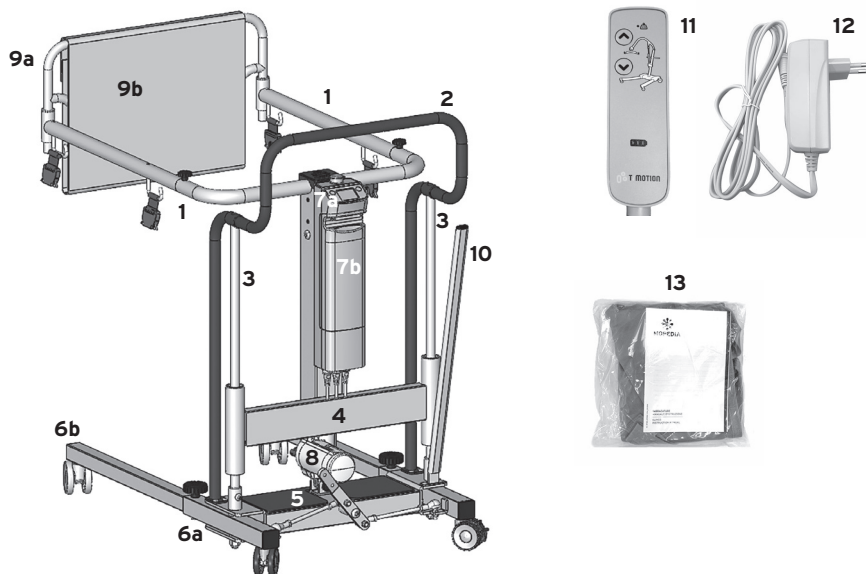
Image for illustrative purpose only

1. n.2 Removable arms equipped with attachment points for harness with locking handwheel equipped with "female" buckle
2. n.1 handle for movement
3. n.2 Sliding columns for lifting
4. n.1 Central body (sliding groups with bearings, support beam, support column, half-arms for patient support)
5. n.1 Basement equipped with:
 - n.1 Non-slip Mat
 - n.1 lever holder tab
 - n.1 rotary selector with 2 levers for leg opening
6. n.2 Legs equipped with:
 - rotating fixed part (6a) with n.1 castor \varnothing 65mm with brake (rear part)
 - removable part (6b) with n.1 castor \varnothing 75mm with brake (rear part)
7. n.1 Control unit (7a) with removable battery (7b)
8. n.1 Lifting actuator
9. n.1 backrest (9a) with n.1 backrest padding (9b)
10. n.1 steel operating lever for leg opening
11. n.1 Remote control
12. n.1 Battery charger
13. n.1 RIA201 Standard harness/seat complete with straps with buckle and loop for height adjustment

7.3 Description of the parts of RIA910 accessory arm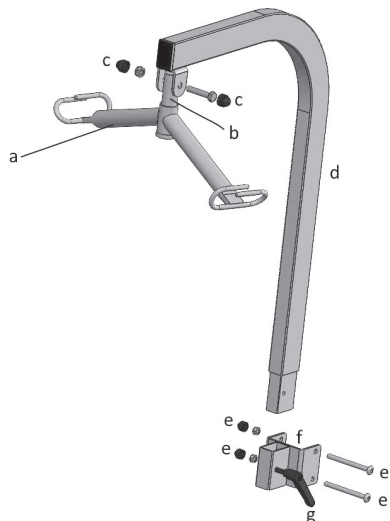





- a. 4 hooks slingbar
- b. slingbar support pin
- c. pin screw b. with nut and caps nut cover
- d. lifting arm
- e. screws with data and nut cover to install RIA910 accessory on RI900-RI905 central column
- f. plate to install the RIA910 accessory on RI900-RI905 central column
- g. arm clamping handle

Image for illustrative purpose only

8. ASSEMBLY INSTRUCTIONS

8.1 RI900-RI905 assembly instructions

1. Open the packing carton and check that the contents correspond to what indicated in the manual and that the parts have not been damaged during transport. In case otherwise, contact your dealer immediately.
2. Take the patient support arms with the relative handwheels (fig.a); if the hooks of the patient's horizontal arm are not equipped with the female part of the buckle, insert the female buckles equipped with the strap with slots on the hooks using the special slot as shown in fig. a1.
3. Insert the patient's support arms equipped with the female buckle (as described in point 2) on the fixed part of the support handle (fig.b). Tighten the locking knobs to fix the arms (fig.c).
4. Insert the backrest into the vertical bushings located at the end of the arms, pushing it down as far as it will go (fig.d). Equip the backrest with padding; wrap the backrest and close the padding with the Velcro (fig.e).

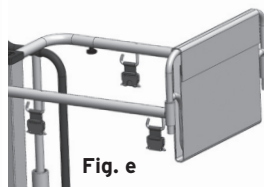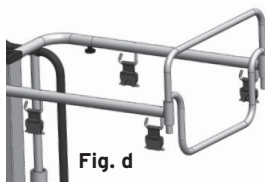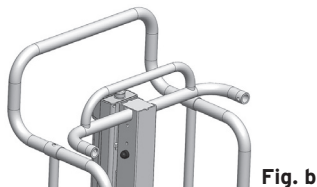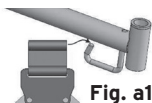5. Install the battery in the appropriate housing of the control unit (fig. f).


Images for illustrative purposes

Fig. f

The device is ready for use.

8.2 RIA910 accessory Assembly instructions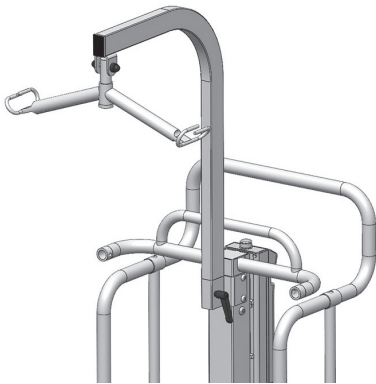

1. Open the packing carton and check that the contents correspond to what indicated in the manual and that the parts have not been damaged during transport. In case otherwise, contact your dealer immediately.
 2. Remove the hole caps on the appropriate holes in the central column and install the plate on it (ref. f) using the appropriate screws (ref. e)
 3. Assemble the lifting arm (ref. a, b, c)
 4. Mount the lift arm on the special housing in the plate e tighten using the clamping handle (ref. g)
- The accessory is ready for use.
Image for illustrative purpose

Note: for optimal use of the RIA910 lifting arm accessory, it is advisable to remove the RI900 lifting half-arms (ref. 1) by unscrewing the special tightening handwheels and pulling out the half-arms themselves; this will allow a better management of the lifted person by the operators, both for the operations to be performed on him and for his handling.

8.3 Assembly instructions for slings

The RIA201 and RIA211 slings in the MOPEDIA line by Moretti are suitable for use with Nuevo Home RI900-RI905 transfer and lifting device.

The RIA201 and RIA211 harnesses are equipped with 4 attachment points (2 for each side) with adjustable heights using buckles and loops which, by sliding them on the straps, allow you to reach different adjustment positions of the person's position and height from the ground with the harness/seat. If the hooks of the horizontal arms of the RI900-RI905 device are not equipped with the female part of the buckle or if it is necessary to replace them, insert the female buckles with the slotted strap on the hooks through the special slot as shown in the figure below (Fig.i).

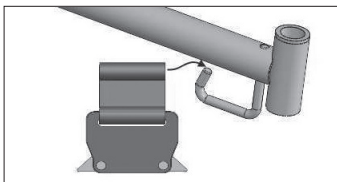
Fig. i

Images for illustrative purpose

Attach the attachment points of the harnesses (buckles with “male”) to the corresponding “female” buckles on the horizontal arm according to the following couplings: **a with A, b with B, c with C, d with D** (Figs. ii, iii, iv, v, vi)

Fig. ii

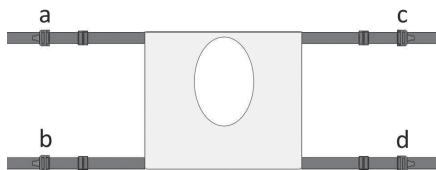
Fig. iii

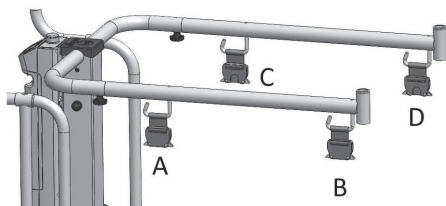
Fig. iv

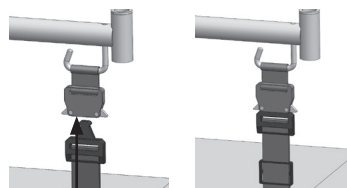
Fig. v

Fig. vi

Image for illustrative purposes

WARNING!

Confirmation of the correct installation of the harness buckles is the ‘click’ when the male buckle is inserted into the female. Also check manually, by pulling the straps downwards, that all the buckles are correctly fitted.

WARNING!

Do not move the patient without the backrest installed on the R1900-R1905 device! Always take care, before moving the person, to check the correct installation of the seat by checking the correct fastening of the buckles and the backrest by checking the complete sliding of the lower tube on the end bushing of the arm.

9. INSTRUCTIONS FOR USE

9.1 Instructions for use of the R1900-R1905 patient lift

1. Remove the backrest by pulling it out of the vertical bushings of the arms by pushing it upwards.
2. Approach the person to be lifted and position the lifter as close as possible to the person to be lifted, possibly opening the legs of the device if necessary.
3. Before lifting, lock the rear wheel brakes.
4. Keep the person centered between the legs of the base and always facing the caregiver.
5. Carefully hook the harness attachment points to the hooks on the lift arms (see par. 8.3).
Confirmation of the correct installation of the harness buckles is the ‘click’ when the male buckle is inserted into the female. Also check manually, by pulling the straps downwards, that all the buckles are correctly fitted.
6. Install the removed backrest taking care to slide down the vertical tubes fully to the stop on the terminal bushes of the horizontal arms.
7. Operate the patient lifting actuator using the appropriate “▲” button on the control unit or on the remote control, lifting the patient to the necessary height, keeping him in the frontal position towards the caregiver.
8. Unlock the rear wheels and transfer the patient.
9. Before starting the descent of the harnessed person, lock the rear wheels again.
10. Activate the patient lowering actuator using the appropriate “▼” button on the control unit or on the remote control, lowering the patient to the necessary height.

NOTE: During normal operation of the lift it is possible to open or close the legs of the lift electrically using the special keys on the control unit and remote control dedicated to opening and closing the legs. The opening or closing operation legs can be performed both with the harnessed patient (with load) and without patient (without load).

WARNING!

ALWAYS CHECK THE CORRECT SITTING POSITION OF THE LIFTED PERSON DURING THE WHOLE MOVEMENT!

During the descent, avoid that the patient may encounter an obstacle that prevents the downward movement of the same. In that case, let the patient go up again and remove the cause of the impediment of descent.

9.1.1 Specific instructions for RI900 Opening/Closing legs


To open or close the legs, operate the leg opening actuators using the buttons at the bottom of the remote control.

The legs can be opened and closed even with the patient in the harness.

9.1.2 Specific instructions for RI905 Opening/Closing legs

To manually open or close the legs, remove the lever possibly located in the lever-holder tab of the base, insert the lever in the central selector and manually rotate it clockwise (opening) and anticlockwise (closing); once used it is possible to remove it again and reinsert it in the tab. The legs can be opened and closed even with the patient in the harness.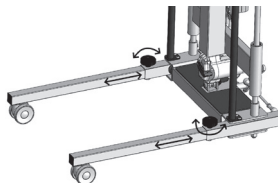

Leg disassembly/assembly

Unscrew and remove the through handwheel that joins the fixed part to the removable part of the legs; remove the detachable part of the leg. To restore the assembly of the leg, position the mobile part of the leg partially inside the fixed part of the leg, and insert the handwheel with the stem in the appropriate leg hole, screwing it through the welded nut in the lower part of the leg.

9.2 Instructions for use of RI900-RI905 patient lift with RIA910 lifting arm

1. Remove the backrest by pulling it out of the vertical bushings of the arms. For use with the RIA910 accessory, it is recommended to remove the horizontal lifting arms by unscrewing the locking knobs and extracting the arms.
2. Approach the person to be lifted and position the lifter as close as possible to the person to be lifted, possibly opening the legs of the device if necessary
3. Before lifting, lock the rear wheel brakes
4. Keep the person centered between the legs of the base and always facing the caregiver

5. Carefully fasten the attachment points of the harness to the hooks on the lifting sling
6. Activate the patient lifting actuator using the appropriate “▲” button on the control unit or on the remote control, lifting the patient to the necessary height
7. Unlock the rear wheels and transfer the patient
8. Before starting the descent of the harnessed person, lock the rear wheels again
9. Activate the patient lowering actuator using the appropriate “▼” button on the control unit or on the remote control, lowering the patient to the necessary height.

NOTE: During normal operation of the lift it is possible to open or close the legs of the lift electrically using the special keys on the control unit and remote control dedicated to opening and closing the legs. The opening or closing operation legs can be performed both with the harnessed patient (with load) and without patient (without load).

 **WARNING!**
ALWAYS CHECK THE CORRECT SITTING POSITION OF THE PERSON LIFTED DURING THE ENTIRE MOVEMENT!

During the descent, avoid that the patient may encounter an obstacle that prevents the downward movement of the same. In that case, let the patient go up again and remove the cause of the impediment of descent.

9.3 Feature Details

AUTOMATIC ANTI-CRUSHING FUNCTION

The electric kit features an automatic patient anti-crushing function which blocks the descent of the lifting arm in the event of accidental contact between the patient and an obstacle.

EMERGENCY STOP

On the control unit there is a red “STOP” button (ref. 1) which, if pressed, immediately deactivates all commands. To restore operation, turn the red knob clockwise until it snaps up and returns to the rest position.

Images for illustrative purpose

EMERGENCY COMMANDS

For RI900 model. There are four buttons on the control unit (two for ascent / descent and two for opening / closing the legs) that can be used in place of those on the remote control to carry out operations in case of operating problems of the remote control itself:

- Button 3: ‘up’ button for lifting
- Button 4: leg opening button
- Button 5: button to close the legs
- Button 6: ‘down’ button for lowering


Images for illustrative purpose

EMERGENCY COMMANDS

For RI905 model. There are two buttons on the control unit that can be used in place of those on the remote control to carry out operations in case of operating problems of the remote control itself:

- Button 1: red emergency button
- Button 2: ‘up’ button for lifting
- Button 3: ‘down’ button for lowering

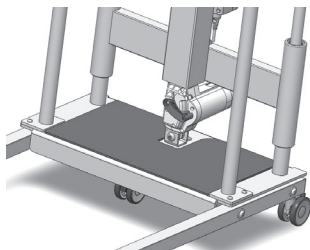MECHANICAL MANUAL LOWERING OF THE ACTUATOR

In case of use of the manual / mechanical descent, make sure that it is carried out in conditions of total safety for the patient. To activate the manual / mechanical descent, pull the red lever in the lower part of the actuator upwards and safely accompany the lowering of the device. Once lowering is complete, return the red lever to the rest position.

INDICATION OF THE BATTERY STATE OF CHARGE

There is a 3 LED indicator for the battery charge status. This indicator is located both on the control unit (only for RI900, ref.2 on the "RI900" picture above) and on the remote control (ref.2 on par. 9.4.1 and ref.4 on par. 9.4.2). Three lit LEDs indicate a complete charge of the battery, while a single lit LED indicates that the battery is low and we need to recharge it.

HOW TO CHARGE THE BATTERIES

- Connect the adapter to the mains socket and then to the input of the control unit after removing the relative protective cap
- There is an orange LED on the charger adapter which is orange during the charging phase. Charge the battery until the LED turns green
- Before proceeding with the use of the lift remember to always disconnect the adapter to recharge the batteries!
- Charge the batteries for at least 6 hours before using the lift

9.4 Instructions for using the remote control

9.4.1 RI900 Remote control

The remote control is connected to the control unit via an extendable spiral cable.

Buttons on the remote control:

1. 'up' button for lifting
2. 'down' button for lowering
3. leg opening button
4. leg closing button

Signaling on the Remote control:

5. LED for overload indication: the lifter has an overload control device that stops movement if the excessive load threshold is exceeded and generates both a visual and acoustic signal
6. LED for signaling the state of charge of the battery

Image for illustrative purposes

9.4.2 RI905 Remote control

The remote control is connected to the control unit via an extendable spiral cable.

Buttons on the remote control:

1. 'up' button for lifting
2. 'down' button for lowering

Signaling on the Remote control:

3. LED for overload indication: the lifter has an overload control device that stops movement if the excessive load threshold is exceeded and generates both a visual and acoustic signal
4. LED for signaling the state of charge of the battery

Image for illustrative purposes

10. WARNINGS

- The lift is a device that allows you to make short transfers of a patient with movement difficulties such as from a bed to a wheelchair or a comfortable chair or vice versa. The lift must not be used for patient transport.
- When lifting the person, make sure that the rear wheel brakes are locked.
- When transferring, avoid driving over obstacles that could cause you to tip over.
- With the patient harnessed on the seat on the device, move the lift only after checking that the backrest is correctly installed.

WARNING!

Always check the good condition of the harnesses.
Do not use slings that show signs of damage or wear.
The Muevo Home patient lift is not intended for use in direct contact with liquids (although it has an IP54 degree of protection), therefore pay particular attention to the protection of the on-board electronic equipment from jets of water or other liquids.

11. CLEANING

For harnesses and back padding, follow the guidelines below:

Maximum washing temperature 30°C; normal process

Do not bleach

Do not tumble dry

Do not iron

Do not dry clean

12. MAINTENANCE

It is extremely important to check the lift before each use:

- Check that all screws and bolts are well tightened, and that the harness, straps and buckles are intact.
- Check that the lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes works correctly.
- The mechanical parts must be lubricated with a drop of oil in order to help smoothness and wear resistance.
- Lubricate as above indicator every two or three months.

13. SPARE PARTS AND ACCESSORIES

For spare parts and accessories, please always refer to the MORETTI master catalogue.

Accessories

Cod.RIA201 Standard harness (1 pcs supplied together with the RI900-RI905)

Cod.RIA211 Harness for WC/toilet

Cod.RIA910 Central lifting arm with 360° rotating 4 hooks sling bar

If you purchase the RIA910 accessory, you can use the following models of harnesses for MUEVO patient lifts to be applied to the slingbar with which the accessory is equipped:

Cod.RP881 Standard harness with 'basic' headrest

Cod.IM100-IM101 Standard harness without headrest

Cod.IM110-IM111 Standard harness with headrest

Cod.IM120 Containment harness with splints

Cod.IM140 Toilet harness without headrest


Cod.IM141 Toilet harness with headrest

Cod.IM150 Containment harness for amputees

ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

14. TECHNICAL FEATURES

RI900 structure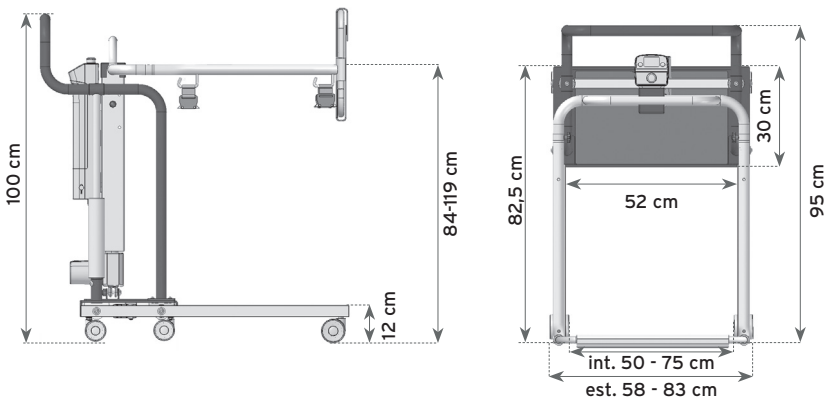


Total weight with structure and electric kit: 57 Kg (with battery installed)

Max capacity (patient weight): 180 Kg

With the supplied harnesses (RIA201, RIA211), it is possible to pick up a person starting from a height of about 25 cm from the ground.

RI905 structure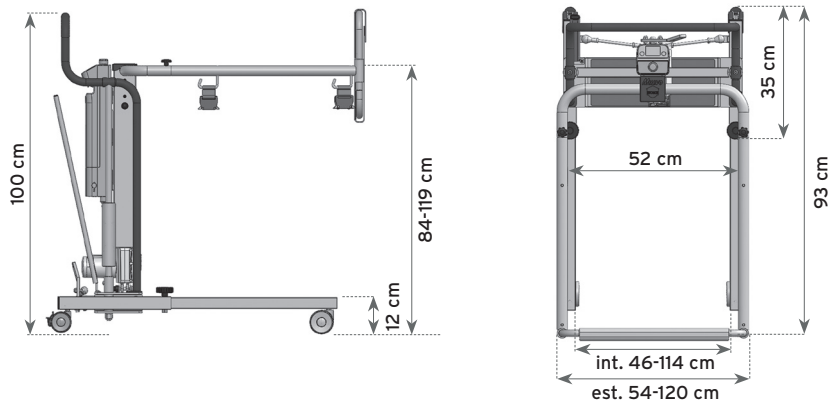


Total weight with structure and electric kit: 39 Kg (with battery installed)

Max capacity (patient weight): 135 Kg

With the supplied harnesses (RIA201, RIA211), it is possible to pick up a person starting from a height of about 25 cm from the ground.

RI900-RI905 structure height with accessory RIA910

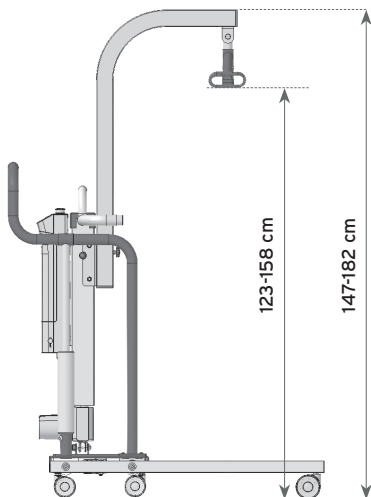The max. load capacity of the RIA910 accessory is given by the device max. capacity (wheter RI900 or RI905) to which it is coupled.

RI900 and RI905 operating temperature range: +5°C / +45°C

15. CONDITIONS OF DISPOSAL

When disposing of the device never use the normal return systems of municipal solid waste. It is recommended instead to disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

15.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

15.2 Treatment of dead batteries (Directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

16. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

16.1 Warranty for rechargeable batteries (if provided)

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

16.2 Reparations

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the repair is under warranty. A repair or replacement does not renew nor extend the warranty.

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

16.3 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

16.4 Exoneration clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

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