MOPEDIA®

ITALIANO

MA R1840_R1841 01 C_ITA_04-2024

METTI IN PIEDI MUEVO UP

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INDICE

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Via _____ Località _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN ITALY**

NOTE

MOPEDIA®

MA R1840_R1841 01 C_ENG_04-2024

STAND UP LIFT MUEVO UP

INSTRUCTION MANUAL

ENGLISH

INDEX

1. CODES	PAG.3
2. INTRODUCTION.....	PAG.3
3. INTENDED USE.....	PAG.3
4. DECLARATION OF CONFORMITY	PAG.3
5. GENERAL WARNINGS	PAG.4
6. SYMBOLS	PAG.4
7. GENERAL DESCRIPTION.....	PAG.4
8. DESCRIPTION OF THE PARTS	PAG.5
8.1 Stand up lift frame	pag. 5
8.2 Electrical kit	pag. 5
9. ASSEMBLY INSTRUCTIONS	PAG.6
9.1 Frame assembly	pag. 6
9.2 Electrical Kit Assembly.....	pag. 6
10. BEFORE USE	PAG.8
11. INSTRUCTIONS FOR USE	PAG.8
12. MAINTAINANCE.....	PAG.10
13. CLEANING AND DISINFECTION	PAG.10
13.1 Cleaning.....	pag. 10
13.2 Disinfection.....	pag. 10
14. CONDITIONS OF DISPOSAL	PAG.11
14.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE.....	pag. 11
14.2 Treatment of dead batteries (directive 2006/66/CE)	pag. 11
15. SPARE PARTS ACCESSORIES	PAG.11
16. TECHNICAL FEATURES.....	PAG.11
16.1 Technical specifications	pag. 11
16.2 Frame dimensions.....	pag. 12
17. WARRANTY	PAG.12
17.1 Warranty for rechargeable battery.....	pag. 13
17.2 Repairing	pag. 13
17.3 Spare parts.....	pag. 13
17.4 Exempt clauses	pag. 13

CE I Class Medical Device

REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL
of 5 April 2017 concerning medical devices

1. CODES

R1840 - Electrical stand up lift max load 200 Kg with legs opening pedal

R1841 - Electrical stand up lift max load 200 Kg with legs opening lever

2. INTRODUCTION

Thank you for choosing a patient stand up lift MUEVO UP by MOPEDIA range, manufactured by MORETTI S.p.A. Their design and quality are a guarantee of comfort, safety and reliability. MUEVO UP by MOPEDIA stand up lifts by MORETTI S.p.A. are designed and built to meet all your demands for a practical, correct use. This user manual provides for some suggestions as to how correctly use the lift you have chosen and offers a lots of valuable advices for your safety. Please read carefully through the manual before using the patient lift/stand up. Should you have any queries, please contact your retailer for advice and assistance.

3. INTENDED USE

Patient stand up lift MUEVO UP by MOPEDIA is intended to stand up and move people with motor difficulties. An assistant operates the machine. The stand up lift cannot be used by one person only.

If you should have any doubts please contact your dealer who will help and advice you correctly.

WARNING!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.P.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of PATIENT LIFTS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

To guarantee greater safety, Moretti SpA followed the EN ISO 10535:2006 guidelines relating to lifters.

5. GENERAL WARNINGS

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The useful life of the accessory is determined by wear of parts that cannot be repaired and/or replaced.
- Be careful when children are nearby.
- The user and/or the patient will have to report any serious accident that have occurred related the device to the manufacturer and appropriate authority of the State which the user and/or patient belongs to.

WARNING!

- Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

6. SYMBOLS

	Product code
	Unique Device Identification
	CE mark
	Manufacturer
	Batch Lot
	Read the instruction manual
	Medical Device
	Conditions of disposal
	Attention, should read the instructions
	Production dates
	Waste Electrical & Electronic Equipment (WEEE)
IP54	Protection against dust and water splashes
	B Type
	Maximun loading weight

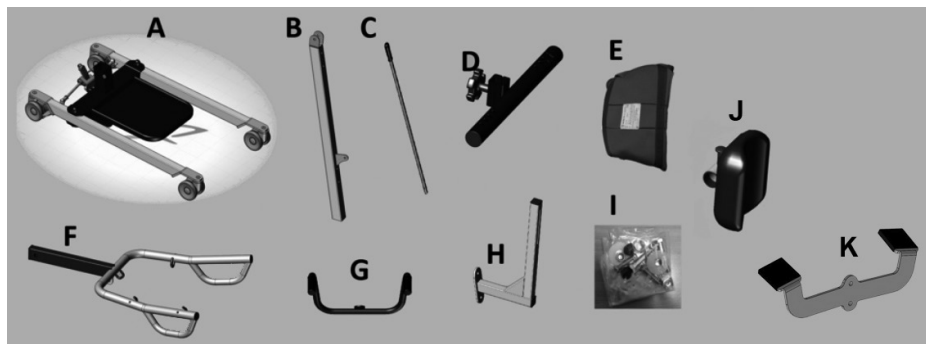
7. GENERAL DESCRIPTION

The electric stand up lift MUEVO UP helps people with motor difficulties to stand up or to be transferred. Ideal for wheelchair users, to stand up. It is the perfect aid to transfer and to stand up the patient from a sitting position with maximum load of 180 kg.

Perfect for those people who need assistance to stand up. Thanks to the width of the opening base the professional work is much easier and the transfers safer. The feet support is fixed and covered by a non-slip felt. The support for the legs is height adjustable. The control unit is provided with security systems. Thanks to its control unit the electric stand up lift MUEVO UP guarantees a comfortable and safe verticalization: the control unit monitors the battery charge level and the actuator conditions.

8. DESCRIPTION OF THE PARTS


8.1 Stand up lift frame

(images for illustrative purposes only)

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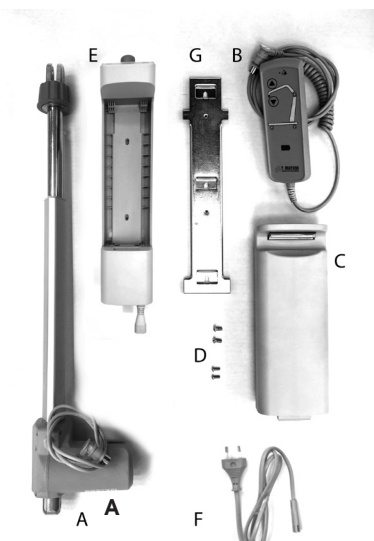(images for illustrative purposes only)

9. ASSEMBLY INSTRUCTIONS

9.1 Frame assembly

1. Unscrew the screw at the base of the column. (Fig.1)
2. Fit the column into the provided cavity of the base. Fit the screw that blocks the column to the base and screw it well. (Fig.1)
3. Insert the bushing in the arm hole arranged for the joint fixing with column (Fig.2a)
4. Connect and Fix the joint between column and arm with the provided screws (Fig.2b)

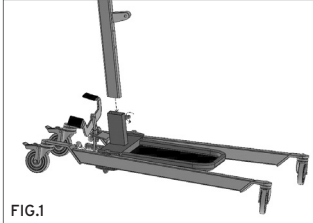
FIG.1

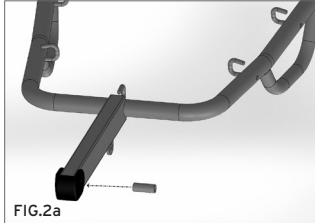
FIG.2a

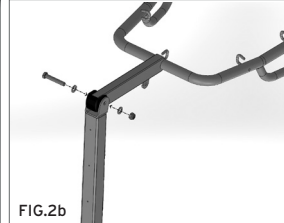
FIG.2b

(images for illustrative purposes only)

5. Position and Fix the Handle on the column with the provided screws. (Fig.3)
- 6a. Fit the pedal for base legs opening and block it with the provided screw. (Fig.4a -for R1840 model)
- 6a. Fit the lever for base legs opening and block it with the provided screw. (Fig.4b - for R1841 model)

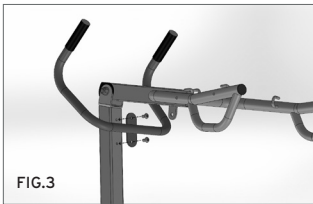
FIG.3

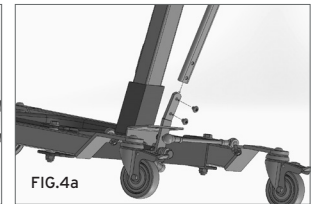
FIG.4a

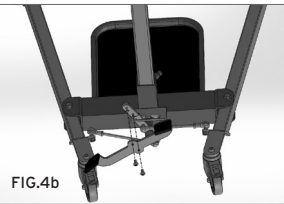
FIG.4b

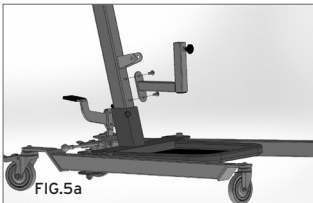
FIG.5a

FIG.5b






(images for illustrative purposes only)

7. Position and Fix the knee support structure on the column with the provided screws; then fit the knee supports, adjusting their height by the provided knob (Fig. 5a-5b)
8. The structure will be completely assembled. Now install the electrical kit for lifting. (see §9.2).

9.2 Electrical Kit Assembly

1. Remove the bolts prepared from the arm and from the column holes to fix the actuator. Fit the actuator in the correct position and fix it using the bolts previously removed. (Fig.6a-6b)
2. Fix the support bracket of the control unit to the column with the provided screws, screwed up the correct fitted holes. (Fig.7)
3. Fit the control unit on the bracket, letting it slide from top to bottom and blocking it with the provided screws. (Fig.8a-8b)
4. Fit the battery into the control unit cavity. (Fig.9a-9b)

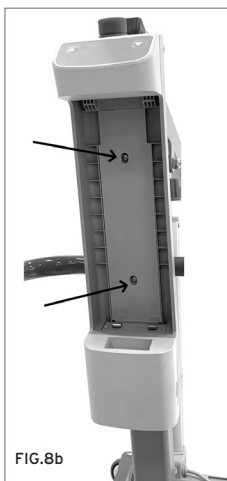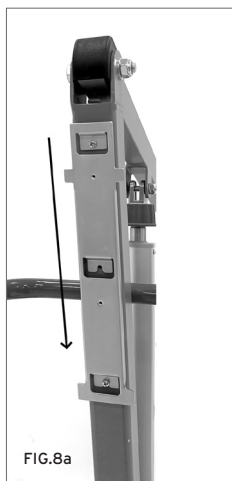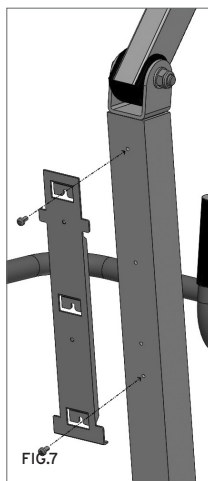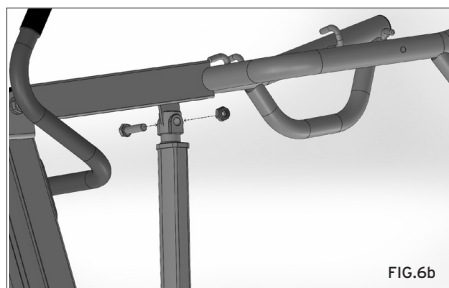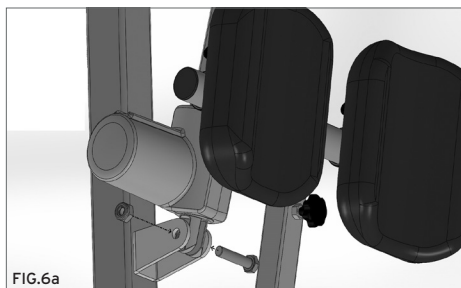5. Connect the actuator connector to the central socket of the control unit and the remote control to the right socket. (Fig.10). The electric kit is correctly mounted and the stand-up lift is ready to use. (Fig.11)

(images for illustrative purposes only)

ENGLISH

Note: before use, be careful to check that the emergency button is not pressed, and if activated, release it to allow the switch on of the control unit.

10. BEFORE USE

MUEVO by MOPEDIA stand up lifts are intended to move people with motor difficulties for short transfers as from a bed to a wheelchair or a commode chair and vice versa. The stand up lift cannot be used to transport the patient.

- Before lifting a patient make sure that the base legs are opened to the maximum width possible, that the opening lever or opening pedal are secured in the opened position and that the rear wheels brakes are blocked.
 - While moving the patient, keep an eye out for obstacles that might cause the lift to tip over.
- ⚠ Always check that the sling is in good condition. Do not use slings showing signs of damage or wear

11. INSTRUCTIONS FOR USE

BEFORE USING THE ELECTRICAL LIFT YOU ARE RECOMMENDED TO CHARGE THE BATTERY FOR ABOUT 6/8 HOURS. AFTER THAT MAKE SURE THE BATTERY IS CHARGED BEFORE EVERY USE.

Before lifting a patient, open the base legs to the maximum width possible:

- slightly pulling the pedal for legs opening towards you with foot, then rotating it to left and locking it in the open position
- slightly pulling the lever for legs opening towards you, then rotating it to left and locking it in the open position

Carry out the same operations in reverse order to bring the base back to the closed status.

1. Block the rear wheel brakes.
2. Keep the patient centered between the base legs and always turned towards the assistant.
3. Adjust the leg support position in height and width using the regulation knobs.
4. Put the sling on the patient, adjusting the abdominal band and the underarm belts.
5. Hook carefully the sling's straps to the lifting beam, see picture (The central hook is to be used with the pelvic sling. The middle hooks are to be used by persons < 1.70m tall. The hooks at the ends of the arm are to be used by persons > 1.70m tall)

NOTE: A pelvic sling is recommended for patients with a sturdy build or low mobility. Thanks to the pelvic belts the underarm pressure decreases and the verticalization becomes more comfortable for the patient

(images for illustrative purposes only)

NOTE: If necessary, for tall persons you can hang the sling on the arm middle hooks, passing the belts through the ends hooks. In this case, the underarm belts become shorter and let the patient's chest approach the arm and incline toward it.

6. Press the up-facing arrow key to lift the patient.
7. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

8. Support the patient until the desired height, keeping him facing his assistant.
9. Release the rear wheels and move the patient.
10. Before lowering the patient, block the rear wheels again.
11. To lower the patient, press the down-facing arrow key.
12. ALWAYS MONITOR THE PATIENT'S POSITION THROUGHOUT THE LIFTING PROCESS!

WARNING!

- Always help the patient during the lowering process so as to maintain the correct seated position!
- While lowering the patient, please remove any barrier that can impede the downward movement. If the case, lift up the patient again and remove what causes the impediment for lowering.

Automatic anti-crushing system

The electrical kit is equipped with an automatic anti-crushing system, which stops the lowering of the lifting boom if there is an accidental contact between the patient and any barrier.

Emergency stop

The control unit is equipped with a red "STOP" button which instantly deactivates all controls when pressed. To restore normal operation, turn the button clockwise and pull it outwards.

Mechanical lowering of the actuator

If using the manual/mechanical lowering, please make sure that it will be done in total safety conditions for the patient. In order to activate the manual/mechanical lowering, pull downwards the red ring placed in the upper part of the actuator and turn it counterclockwise. Once finished the lowering process, take the red ring back to its resting position.

Emergency keys

On the control unit there are two keys (one with an up-facing arrow and the other one with a down-facing arrow) that can be pressed instead of those ones on the remote control to complete all the operations when the remote control fails to work properly.

(images for illustrative purposes only)

Indication of the battery state of charge

There is a 3-Led indicator for the battery state of charge. This indicator is placed on the remote control. 3 Leds on indicate a complete charge of the battery, whereas only one Led on indicates the battery is out of power and needs to be charged.

Recharging batteries

- Connect the charging cable to the main socket and then to the socket on the control unit.
- On the remote control there is a battery charge indicator which flashes during the battery charging phase; once all three LEDs are lit steadily, the battery is fully charged.
- Before activating the stand up lift again, make sure to disconnect the battery charger cable!
- Charge the batteries for at least 6 hours before using the stand up lift.

**ATTENZIONE!**

During charging operations:

- The emergency stop button must remain inactive (therefore control unit active, not blocked, button not pressed); with the emergency button pressed it does not allow the battery to be recharged.
- The system does not allow any movement of the actuator and therefore of the lifting arm for safety reasons.

Instructions for use of Remote Control

The remote control is connected to the control unit by a spiral extendable cable.

Keys:

1. „UP“ key for lifting
2. „DOWN“ key for lowering

Visual indications:

3. LED for max. load exceeded: the lift movement is interrupted if the load exceeds the max. load threshold and the control unit generates an acoustic signal.
4. Led for monitoring and signaling the battery charge status

12. MAINTENANCE

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

It is extremely important to check the lift before each use:

- More specifically, please check that all screws and bolts are well tightened.
- Please, check the lift can move freely with the wheels unlocked by the brakes.
- Check the rear wheel brakes function correctly.

It is very important to regularly check the connection pins and stop rings are correctly located (the pins connect the actuator to the upper and lower part of the lift, and the lifting beam where you hook the cradle).

The mechanical parts must be lubricated with a drop of oil in order to facilitate their sliding and resistance to wear. In particular, pay attention to the mechanical retractor for the legs of the patient lift. Lubricate as indicated above every two or three months.

13. CLEANING AND DISINFECTION**13.1 Cleaning**

To clean the cradle, handle and the structure use a solution of alcohol and neutral soap (in equal parts) mixed with water.

13.2 Disinfection

If you need to disinfect the product use a common disinfectant neutral detergent.

Note: Never use acids, alkalines or solvents such as acetone or diluent. Keep away from light, moisture, heat.

14. CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

14.1 Warnings for the proper disposal of the product, as required by the European Directive 2012/19/UE

At the end of its service life, the product must not be disposed of together with normal municipal waste. It must be delivered to the municipal sorted waste collection centre or to the resellers, in case they provide for this service. Sorted waste disposal reduces the environmental and health consequences of an improper disposal. Furthermore, the materials which make up the product may be recycled, thus resulting in energy and resource savings. In order to highlight the compulsory sorted disposal of the electro-medical equipment, the product itself bears the barred bin symbol.

14.2 Treatment of dead batteries (directive 2006/66/CE)

The symbol on the product indicates that the batteries must not be treated like common household waste. Make sure the batteries are properly disposed; this prevents the possible environmental and health consequences of an improper disposal. Recycling helps to preserve natural resources. Deliver dead batteries to a proper recycling centre. For more detailed information regarding the disposal of dead batteries or of the product itself, please contact the municipality, the local waste disposal centre or the reseller from whom you purchased the device.

15. SPARE PARTS ACCESSORIES

- RP893** Body harness with dorsal abdominal band with Velcro closure
- RP893XL** Body harness with dorsal abdominal band with Velcro closure - size XL
- RP896** Body harness with dorsal abdominal band with Velcro closure and pelvic belts

For spare parts and accessories, please always refer to the MORETTI master catalogue.

ONLY USE ORIGINAL MORETTI ACCESSORIES. MORETTI S.P.A. COMPANY SHALL NOT BE LIABLE FOR ANY INCIDENT CAUSED BY IMPROPER USE.

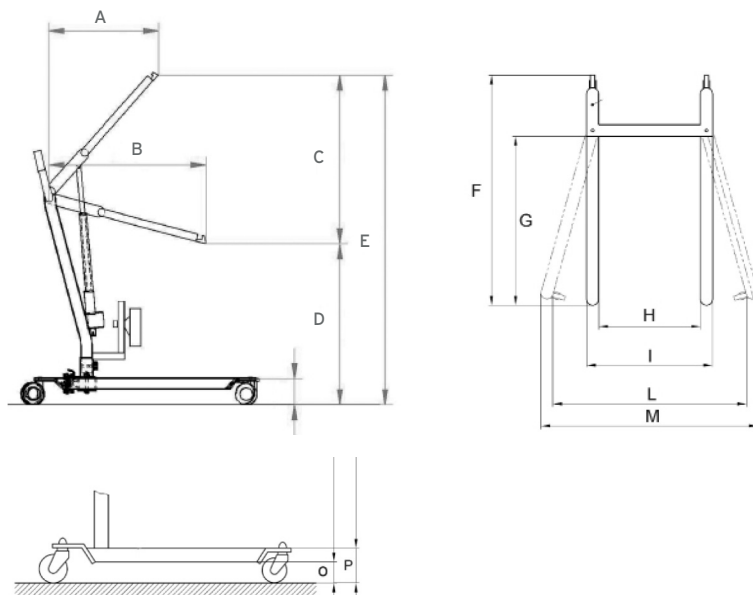
16. TECHNICAL FEATURES

16.1 Technical specifications

- Powder coated steel frame
- Platform covered with felt
- Knee pad adjustable in height and width
- Castors with brake and rotational block
- Leg opening lever (R1841) or pedal (R1840) with safety lock
- Comfortable handle for the healthcare professional
- Hooks designed specifically to hold the sling in complete safety
- Autonomy: over 100 lifts with fully charged battery
- Total weight: 39 Kg
- Maximum loading weight: 200 Kg

16.2 Frame dimensions

ENGLISH

Legend

- A Sagittal dimension at max. verticalization: 2 cm
- B Arm lenght: 80 cm
- C Maximum stroke: 73 cm
- D Minimum arm height: 107 cm
- E Maximum arm height: 180 cm
- F Maximum base lenght: 101 cm
- G Distance between front castors and base mast: 68 cm
- H Minimum internal width: 45 cm
- I Minimum external width: 55 cm
- L Maximum internal width: 84 cm
- M Maximum external width: 94 cm
- O Minimum legs height from ground: 8 cm
- P Maximum legs height from ground: 13 cm

17. WARRANTY

Every Moretti product is guaranteed for 2 (two) YEARS from the purchase date, except the restrictions specified below. The warranty shall not apply in case of damage caused by improper use, abuse, alterations of the product or any other use not in compliance with the user manual. The proper use of the product is indicated in the user manual. Moretti S.p.A. shall not be liable for any damages or personal injuries due to incorrect assembly/improper use of the device, not compliant with the user manual. Moretti does not guarantee its products against damages or faults in the following cases: natural disasters, non authorized maintenance or reparations, damages caused by electrical supply problems (if provided), use of parts or components not provided by Moretti, failure in accomplishing the guidelines and instructions for use, non authorized alterations, damages due to the transport (different from Moretti original delivery) or failure in maintaining the device as indicated in the manual. The components subject to wear are not covered by this warranty if the damage is due to the normal use of the product.

17.1 Warranty for rechargeable battery

The original batteries and the spare batteries are covered by a (ninety) 90-days warranty as far as their performance is concerned and by a (six) 6-months warranty as far as manufacturing faults are concerned or as required by law. Warranty on fully charged batteries left in an unused state for more than (three) 3 consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than (three) 3 consecutive days is automatically void.

17.2 Repairing Warranty repair

If a Moretti product shows material or manufacturing faults during the warranty period, Moretti S.p.A., together with the client, will consider whether the product fault is covered by warranty. Moretti S.p.A., at its unquestionable discretion, may replace or repair the item under warranty in a Moretti reseller point or at its own qualified offices. Labour costs for repairing the product may be at Moretti's expense if it is determined the reparation is under warranty. A reparation or replacement does not renew nor extend the warranty.

Repair not covered by warranty

A product, not covered by warranty, may be returned for reparation only after receiving prior authorization from Moretti client service. Labour and delivery costs for an out-of-warranty reparation will be fully at the client's or reseller's expense. Reparations of out-of-warranty products are guaranteed for 6 (six) months, from the day the client receives the repaired product.

Non defective device

The client is informed if, after examining and testing a returning product, Moretti decides the product is not defective. The product will be sent back to the client and all the return delivery costs will be at his expense.

17.3 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the day you receive them.

17.4 Exempt clauses

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period.

Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exoneration or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice

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